

Directions for Sending Topographies

Medmont

1. Select the map(s) you want to send. More than one map can be selected by holding down the "Ctrl" key.
2. Select "File" and then "Export".
3. Save the file. "Save as Type" should be "Compressed Export File (*.mcf)".
4. Upload the file to www.artoptical.com/ordering

Tomey TMS-4

1. Right click on the map you would like to send.
2. Choose "Slide Making".
3. This will save the map as a .pdf file in whatever folder you choose.
4. Upload the file to www.artoptical.com/ordering

Marco 3D Wave and 3D System

1. Display the map/image that you would like to send.
2. Choose the "Save" icon on the "Map View" screen. The "Save Map Image" dialog opens.
3. Name the file for the map image.
4. Choose an "Output Type", either .jpeg or .bmp format and Save
5. Upload the file to www.artoptical.com/ordering

Topcon

1. Display the map/image you would like to send.
2. Press "Print Screen" on your keyboard.
3. Paste the image into MS Paint or MS Word and choose "Save As".
4. Upload the file to www.artoptical.com/ordering

Topcon CA 800

1. Select patient and exam
2. Choose "Export"
3. Select location (ie. Desktop) and then OK
4. Upload the file to www.artoptical.com/ordering

Humphrey Atlas

1. Display the map or image you would like to send.
2. Go to "Options" and select "Print".
3. Under "Output To" change to "Image File".
4. Under "Output File Type" select .jpeg
5. Click "OK".
6. This brings up the "Print to File" screen. Select location (ie. Desktop) and name the file (ie. Patient X)
7. Upload the file to www.artoptical.com/ordering

Oculus Pentacam or Keratograph

1. Select the patient and exam. One or multiple exams can be exported for a single patient.
2. Click Export
3. Select Single file (U12). Data can be anonymized by clicking the Anonymous datacheck (optional)
4. Click [...] button
5. Choose location (ie. Desktop), file name (ie. Patient X) and Save
6. Upload the file to www.artoptical.com/ordering

ReSeeVit

1. Display the images you would like to send.
2. Right-click on the topography map and choose "Save".
3. Upload the file to www.artoptical.com/ordering

Nidek Magellan

1. Right-click on the topography image you would like to send.
2. Select "Export Map" (at the bottom of the menu).
3. This will create a file that can be saved to your computer either in .jpeg or .bmp format.
4. Upload the file to www.artoptical.com/ordering

ET-800

1. Display the image you want to send.
2. Select "Print Screen" on the keyboard.
3. Paste the image into an MS Word or MS Paint document.
4. Upload the file to www.artoptical.com/ordering

Keratron/Scout

1. In the "Help" section of the software, it will give detailed instructions on sending topography maps.
2. OR...You can display the image you would like to send and choose "Print Screen" on your keyboard.
3. Paste the image into an MS Word or MS Paint document.
4. Upload the file to www.artoptical.com/ordering

Orbscan

1. Display the map that you would like to send.
2. Go to "File" and choose "Save As". Choose .jpg as the file type.
3. Orbscan systems should not be used with the Internet, so it is important to save the file onto a USB drive or disk.
4. Upload the file to www.artoptical.com/ordering



Consultation Hours

Weekdays - 9:00 a.m. to 6:00 p.m. EST

Direct Toll-Free Consultation Line 800.566.8001

Gain immediate access to our Consultation Department through our dedicated Consultation Line.